

WORK INSTRUCTIONS

3.6.2 Qualitative Confirmation of Acidic and Neutral Drug Compounds in Blood

REFERENCE MATERIAL

Mixed Standard Solution

- ✓ Add 20 μ L of each compound.
- ✓ Total volume is 10mL.
- ✓ Solution is stable for 6-months @ room temperature.
- ✓ Record Preparation on standard log.

1mg/mL Stock Solution
Acetaminophen
Butalbital
Carbamazepine
Carisoprodol
Meprobamate
Phenobarbital
Secobarbital

Internal Standard Mix

Proadifen

Prepare 10mg Dilute 'N Shoot Standard with 10mL of DI water.

Prepared standard is stable for 4 months when stored at 4°C.

Working Internal Standard Solution [20ng/ μ L]

Add 200 μ L each Proadifen and 1mg/mL Aprobarbital stock solutions to 10mL volumetric ball flask. QS with DI water.

Solution is stable for three months when stored at room temperature.

WORK INSTRUCTIONS

3.6.2 Qualitative Confirmation of Acidic and Neutral Drug Compounds in Blood

