


Idaho State Police

Forensic Services

Approval for Quality System Controlled Documents

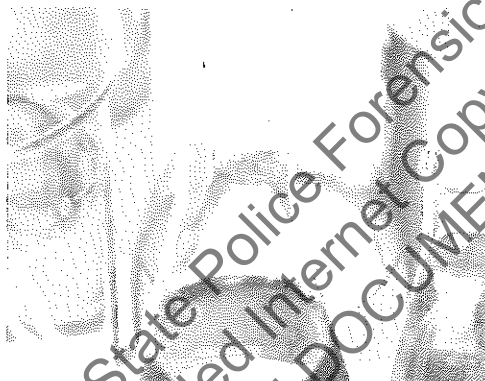

Discipline/Name of Document: Toxicology Work Instructions
2.4.4- Qualitative Confirmation of Carboxy-THC in Urine

Revision Number: 0

Issue Date: 6/29/2007

APPROVED BY: *Courina C. Dursley*
Quality Manager

6/29/07
Date Signed

WORK INSTRUCTIONS

2.4.4 Qualitative Confirmation of Carboxy-THC in Urine

REFERENCE MATERIAL

Standards

Stock Standard Solution

100µg/mL 11-nor-9-carboxy- Δ^9 -THC

Working Standard Solution (1800ng/mL)

Add 900µL Stock Solution to 49.1mL Methanol. Solution is stable for six months when stored at 4°C.

