

## WORK INSTRUCTIONS

### 2.4.3 Qualitative Confirmation of Benzodiazepines in Urine

#### REFERENCE MATERIAL

##### Benzodiazepine Positive Control

- Stock Solutions  
1 mg/mL single component or 250µg/mL multicomponent benzodiazepine class reference solutions.
- Direct spiking  
Spike negative urine with 3µL of each single component stock or 12µL of 250µg/mL stock solution.
- Working Control Solution (10ng/µL)  
(Note: A minimum of two benzodiazepine class compounds must be included in the control solution. One of these compounds must form a TMS derivative.)

Add 100µL each 1mg/mL or 400µL 250µg/mL Stock Solution to ≈ 9mL MeOH in 10mL ball flask. Bring up to volume with MeOH. Solution is stable for one-year when stored under refrigeration.

##### Conjugated Controls

- Commercially obtained conjugated control (example: Bio-Rad C3)
- Stock Solution  
1 mg/mL Oxazepam Glucuronide or 100µg/mL Lorazepam Glucuronide or 100µg/mL Morphine Glucuronide
- Direct spiking  
Spike negative urine with 3µL of 1mg/mL stock solution or 30µL of 100µg/mL stock solution.
- Working Glucuronide Solution (10ng/µL)  
Add 100µL of 1mg/mL Stock Solution (or 1mL 100µg/mL Stock Solution) to 10mL MeOH. Solution is stable for one year when stored under refrigeration.

##### Internal Standard

- Stock Solution  
1mg/mL Prazepam.

##### Direct addition

Add 3µL of 1mg/mL stock solution to each 6mL urine sample and control.

- Working Internal Standard Solution

Add 100µL Stock Solution to ≈ 9mL MeOH in 10mL ball flask. Bring up to volume with MeOH. Solution is stable for one year when stored under refrigeration.


#### COMMENTS

This method has instructions for the preparation of both an underivatized and a TMS-derivatized extract. The analyst has the option of preparing either or both of these extracts.

## WORK INSTRUCTIONS

### 2.4.3 Qualitative Confirmation of Benzodiazepines in Urine

#### SAMPLE PREPARATION: Glucuronide Cleaving

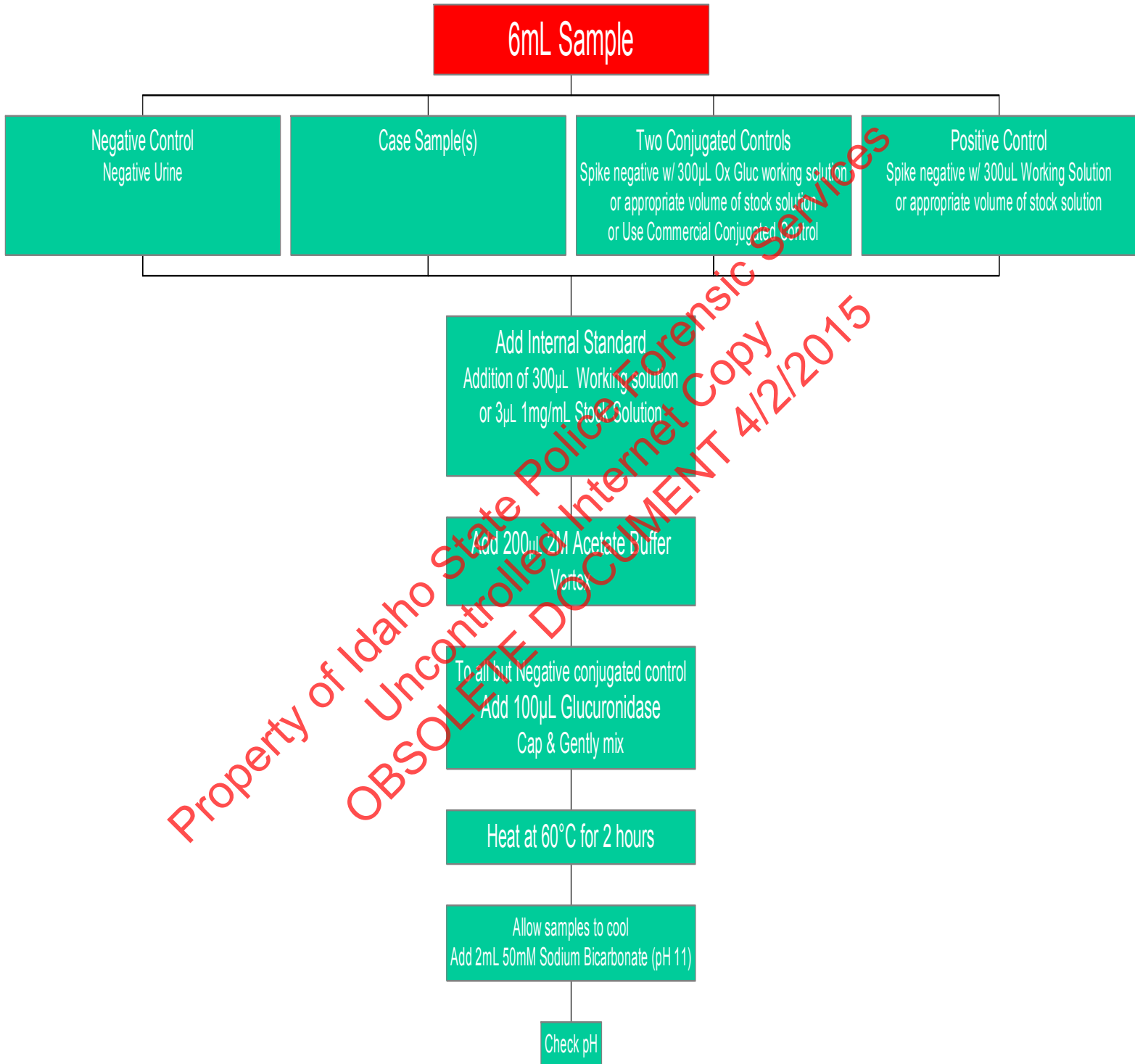

## WORK INSTRUCTIONS

### 2.4.3 Qualitative Confirmation of Benzodiazepines in Urine

#### SAMPLE PREPARATION: Extraction and Reconstitution and/or Derivatization

